## Protac KneedMe<sup>®</sup> To Go

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GB - Directions for use

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SE - Bruksanvisning

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Medical Device

[www.protac.dk](http://www.protac.dk)

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## Protac KneedMe® To Go

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# GB - Directions for use

## Protac KneedMe® To Go

You have chosen a Protac quality product. Protac guarantees that you have acquired an original, Danish design product, developed in cooperation with professional business partners. Since 1994, Protac has been designing, producing and selling sensory-stimulating devices for the health sector in Denmark, Europe, Australia and New Zealand.

Protac's products are developed on the basis of theories about sensory integration and are well documented with respect to research, documentation and case descriptions.

Protac production process focuses on environment and safety. Protac plastic balls are certified STANDARD 100 by OEKO-TEX® nr. 2076-310 DTI.

For further information see [www.protac.dk](http://www.protac.dk)

GB

## Labelling

The products are labeled in accordance with Regulation (EU) 2017/745 Class I medical devices. CE- and care labels with information about the manufacturer, raw materials and washing instructions are placed inside the cover.

Key to label symbols: please refer to Key to symbols

## General precautions - Safety

Please note that Protac KneedMe® To Go is not intended to curtail personal freedom or to be a safety device.

- ⚠ • The shoulder strap of the knee blanket can be used as a belt. This must only be fastened around the waist of the user, never around the back of a chair. The belt prevents the knee blanket from sliding down when being used. When the strap has been adjusted and tightened correctly, it should fit closely around the user, comfortably and without being too tight.
- ⚠ • Users who are unable to unfasten the belt and free themselves from the knee blanket without assistance, should always be supervised.
- ⚠ • If a user's movements or reduced cognitive abilities inadvertently lead to release from the knee blanket, we recommend that the user should also be supervised.
- ⚠ • Take precautions to avoid pressure sores when using Protac KneedMe® To Go.
- ⚠ • Cease using the product if there are any signs of redness or irritation on the user's thighs or buttocks as a result of using Protac KneedMe®.
- ⚠ • Do not use near an open flame.
- ⚠ • Avoid tripping over the shoulder strap/belt.
- ⚠ • When using Protac KneedMe® To Go in a car, it must not be used by the person driving the car.
- ⚠ • If a passenger is using Protac KneedMe® To Go, it must lie loosely over the knees. Do not use the shoulder strap around your waist as a belt.

## Incidents

- Incidents related to the use of Protac KneedMe® To Go must be reported to the manufacturer
- Serious incidents related to the use of Protac KneedMe® To Go be reported to the manufacturer and the competent authority of the country where the incident takes place.

## The product's purpose and application environment

Sensory integration is the basis for the design of all Protac's products. This theory originated in the United States and was developed by A. Jean Ayres, an occupational therapist and psychologist, back in the 1970s. The theory is based on how the brain processes sensory impressions, and how one can influence the various sensory stimuli via weight and tactile sense (the sense of touch) in the skin and the proprioceptive sense (sense of movement) in the muscles and joints. The pressure, movement and weight of the balls have an impact on these senses.

GB

Protac KneedMe® To Go is developed as an aid for people with disabilities.  
Children from the age of 5/6 can use Protac KneedMe® To Go:

- Brain damage and other neurological diseases.
- ADHD and Autism
- Psychiatric diseases
- Developmental disorders, sensory disorders.
- Physical and mental unrest.
- Concentration problems and learning difficulties.

Protac KneedMe® To Go is used by people with brain injuries and other neurological diseases. These patients often suffer from neurogenic disorders. The movements and pressure of the balls give the body sensory stimuli that contribute to an increased body sensation.

For people with ADHD and autism, Protac KneedMe® To Go is used to create calm and concentration. The movements and pressure of the balls on the body provide sensory stimuli that create calmness and help to define the body. The use of the knee blanket enables users to maintain concentration for longer periods.

Protac KneedMe® is used for people with psychiatric illnesses. Many have violent movements and psychotic behavior which subsides when they get the knee blanket over them. Patients become more present in contact with others.

Protac KneedMe® is used for people with eg developmental disorders, sensory disorders to create increased sensory stimulation for the body.

Protac KneedMe® is used for people with physical and mental unrest, such as restlessness in the legs. The movement of the balls in the knee blanket provides sensory stimuli in the legs and an increased body sensation and calmness

## Use of the product

Protac KneedMe® To Go should lie over the user's lap.

Attach the shoulder strap by inserting the snap hook through the first ring and securing it in the second ring.

You can fold up the Protac KneedMe® To Go and carry it by the two handles.

If Protac KneedMe® To Go is slipping off during use, you can use the shoulder strap as a belt to fasten round the waist.

You can fold up the Protac KneedMe® To Go and carry it using the shoulder strap.

If you have a tablet or mobile phone in your Protac KneedMe® To Go, it is a good idea to zip it into the rubber outer section for better protection during transport.

GB

## The product's service life

The product has an expected service life of 5 years.

The service life is based on proper use, cleaning, washing, service and maintenance. Products, which are used for longer than the above-mentioned service life, should be assessed prior to further use. It is the responsibility of the user to check regularly whether the product can be used for longer than the stipulated service life.

## Cleaning

Protac KneedMe® blankets can be washed. The regularity and need for washing depends on use. When washing Protac KneedMe® To Go, follow the instructions below.

- Protac KneedMe® To Go can be washed in a washing machine with a min. capacity of 7kg, using washing detergent without bleach. Spinning at low speed. Also see the washing instructions in the cover.
- The cover and ball bag must be washed separately. The laundry bag supplied is recommended for washing the ball bag.

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring. Protac does not recommend the use of spot remover.

GB

Protac cannot be held liable for incorrect washing and maintenance.

## Maintenance

We recommend airing the knee blanket regularly.

Check your Protac KneedMe® To Go every time you use it for flaws and defects, and once a year inspect the knee blanket's seams, zip, fabric, plastic balls and general condition. If you become aware of a serious flaw or defect, stop using your Protac KneedMe® To Go until the flaw has been remedied and safety restored.

## Transport and storage

Protac KneedMe® To Go is delivered packed in a polybag.

When the knee blanket is not being used, it should be stored safely to protect it against dirt and moisture.

## Disposal

When disposing of a Protac KneedMe® To Go, it can be divided into the following parts: plastic and fabric.

- The plastic balls can be disposed of as hard plastic.
- All parts can be disposed of as combustible waste.
- Waste must be disposed of in accordance with applicable rules.

## Warranty

There is a 2-year warranty for manufacturing defects

## The warranty is void in the following instances:

- Overload of the Protac KneedMe® To Go
- Heavy-handed use of the Protac KneedMe® To Go
- Lack of, or improper maintenance of the Protac KneedMe® To Go

**Traceability:** Production month, year and batch number are marked on the washing label. These must not be removed from the product.

## Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

Washing bowl: Maximum washing temperature (see care label)

Warning:  
See under Safety

Triangle:  
Do not bleach

User information

Square with a circle:  
tumble drying possible  
(see care label)

For indoor use

Iron:  
Do not iron

CE marked according to  
(EU) 2017/745 Class I  
medical devices  
Class I

Circle:  
Do not dry clean

Manufacturer

Batch number

## Technical specifications

**Standards:** Comply with general requirements for aids acc. to the following harmonized standards:

**EN ISO 13485:2016/A11:2021:** Medical Devices – Quality management systems

**EN ISO 14971:2019/A11:2021:** Medical devices - Application of risk management to medical devices

**DS/EN ISO 21856:2022 :** Medical devices - Assistive products - General requirements and test methods.

## Service information

Do not use balls in your Protac KneedMe® To Go other than those used by the manufacturer.

## Guide

- to daily inspection Protac KneedMe® To Go prior to use

- 1 • All parts are in place, no loose parts

GB

- 2 • No evidence of holes, or damage to fabric and stitching

Protac A/S  
Niels Bohrs Vej 31 D, Stilling  
DK-8660 Skanderborg  
Tel.: 86 19 41 03  
e-mail: [protac@protac.dk](mailto:protac@protac.dk)

Distributor list: Please refer to: [www.protac.dk/kontakt](http://www.protac.dk/kontakt)

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